


THERE'S BEEN A BIG CHANGE IN ALLERGY SKIN TESTING.

Multi-Test® PC (pain control) alleviates the pain of skin testing. Here's how it works: the neurological gate-control theory involves the activation of nerve cells that block pain transmission, resulting in pain suppression. Multi-Test PC touch post activates these nerve cells to inhibit pain, while the refined points provide well-defined wheals with very little tissue trauma.

Multi-Test PC provides high sensitivity, high specificity, and low variability in results—with virtually no pain.

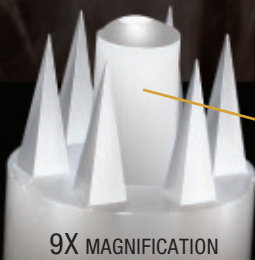

9X MAGNIFICATION

The Multi-Test PC **TOUCH POST** activates nerve cells to inhibit pain.



Multi-Test PC fits into Multi-Test® II extract wells, which are designed to prevent reversing device position.



Allow touch posts to contact skin for one second and press loaded Multi-Test PC **FIRMLY** into skin at test sites.



4X MAGNIFICATION

All test sites should show this impression on the skin with proper pressure.



Multi-Test PC provides easy-to-read and highly reproducible histamine results.

FINALLY, CONTROLLED PAIN WITHOUT SMALL, HARD-TO-READ REACTIONS: MULTI-TEST PC.

Made in USA

Decatur, Illinois • (800) 537-1336 • www.lincolndiagnostics.com
ISO 13485 certified • OSHA compliant • US and Foreign Patents Pending

L I N C O L N

*Diagnos*tics, INC.